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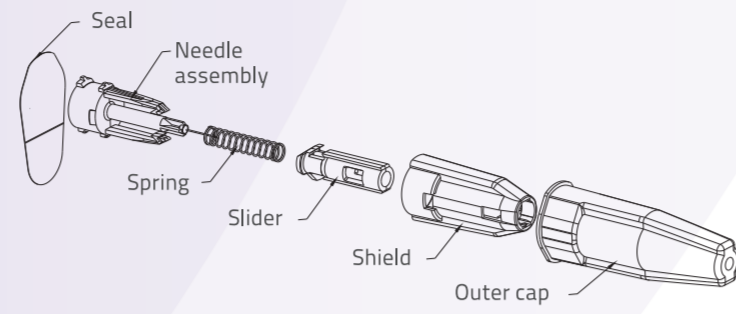
KEY FEATURES

- Automatic needle lockout after use.
- Visual indication of needle lock out via red stripe.
- Needle safely contained behind shield preventing accidental exposure.
- The needle remains hidden from view increasing patient comfort.
- Viewing window enables checking drug flow through the needle.
- Special wing-shaped design for a comfortable and secure grip.
- Available in two most popular needle sizes: 31G x 6 mm and 31G x 8 mm.

INSTRUCTIONS FOR USE

Read the needle gauge and length on the seal to make sure you are using the right safety pen needle for you or ask for assistance if you have impaired vision. Also do so, if you are a healthcare professional, a caretaker or family member assisting a patient. Ask your healthcare professional for assistance in choosing the needle length, injection site and techniques appropriate for you. Recommended injection technique: make an injection into flat skin vertically at 90° (degree) angle. Alternative injection technique: you may also make a skin fold if recommended by your healthcare professional.

- Check the seal. If it is missing or damaged use another safety pen needle. Remove the seal. Check the needle, use another one if it appears to be damaged.
- Push the safety pen needle straight onto the pen holding it in one axis with the pen and twist until it is tight. DO NOT over-tighten.

- Set the prescribed dose.
- Insert the needle into the flat skin at 90° (degree) angle in one continuous movement until the shield stops. Make an injection. DO NOT remove the needle until the dose has been completely delivered. Refer to the instructions for use of the pen injector device manufacturer for specific hold time.

- Remove the outer cap. Make sure not to press the shield before use as any pressure on the shield may cause the needle to lock and the pen needle to be unusable. Use another needle if the red stripe is visible in the viewing window or the needle appears to be damaged.

- When dose delivery is complete, remove the needle from the skin. The shield will automatically cover the needle until it locks. A red stripe will appear in the viewing window. The needle is locked and cannot be reused. Note: Liquid may fall back into the safety pen needle which may result in fluid being left on the skin after injection.

- Check if the needle is visible in the viewing window. Use a new safety pen needle if the needle is not visible in the viewing window.

- Twist off the needle to remove it from the pen injector device.

- Perform a priming test as recommended by the pen injector device manufacturer. A drop of liquid should appear on the needle tip visible through the viewing window. Use a new safety pen needle if the priming test was unsuccessful.

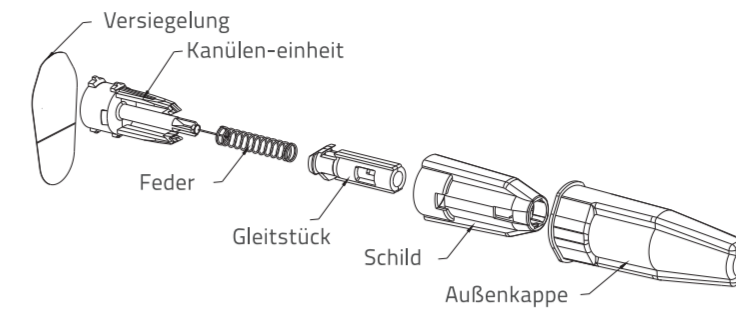
- Discard used safety pen needle into a sharps container.

WARNINGS

- Keep out of reach of children.
- Avoid infections: Before making an injection, clean & disinfect the injection site. Always follow your facility sharps injury prevention guideline or local regulations.
- Do not use if the seal or outer cap is damaged or missing, or if the shield or needle appears to be damaged.
- Refer to the instructions for use of the drug/pen injector device for specific storage and handling instructions.

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